



Agoralaan Abis • 3590 Diepenbeek • Belgium
Tel: +32(0)11/28.69.02 • www.bedal.be

Product data sheet: BEDALPATCH

Reference: 24b-Product data sheet

Product profile


The Bedalpatch is a single-use dressing for covering an exit site of an intravenous catheter during showering, in order to provide a watertight barrier.

Adhesive: skin friendly acrylic adhesive

Release liner: white paper

Film: Transparent Polyurethane film (PU)

Snap part material: Polyvinylchloride (PVC)

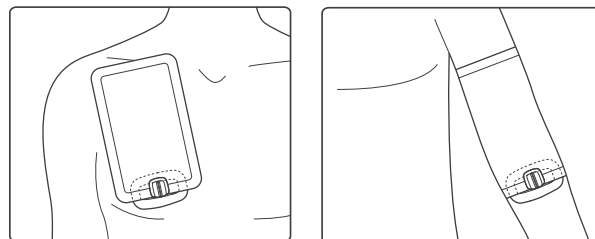

Applications

The Bedalpatch can be used to cover the exit site of intravenous catheters, such as a peripheral catheter, a PICC (peripherally inserted central catheter), midline catheters, CVC (central venous catheters) and implanted ports. The Bedalpatch provides a watertight seal around the connected infusion line with 1 lumen. The dressing is applied before washing, and removed directly after washing.

Technical properties

Property	Value
Thickness film (micron)	36
MVTR film (g/m ² per 24h @ 37°C)	1000
Peel strength between carrier and film (cN/25mm)	3

Adhesive biocompatibility

	Test method	Value
Primary skin irritation index	ISO 10993-10:2010	0,17
Sensitization	ISO 10993-10:2010	Non-sensitizing
Cytotoxicity	ISO 10993-5:2009	Non-cytotoxic

Product features

Well tolerable skin-friendly-acrylic glue with good level of adhesion to skin

Comfortable wear that allows for movement during washing

Transparent patch to allow for inspection of exit site

Product variants

The Bedalpatch is available in 2 variants: the Bedalpatch and the Bedalpatch comfort. The Bedalpatch comfort is smaller in size, and has an acrylic tape with a higher conformity. The bedalpatch comfort has an additional transparent layer on the top patch. This layer provides strength during application of the patch and is removed once the patch is positioned.

Storage

Store the product in dry conditions, at 20°C ± 10°C and in its original packaging

Use the product within 36 months after delivery

Version V3, April 29, 2016