

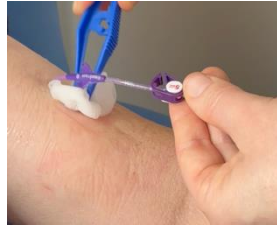


PROCEDURE PLACING A FLEXGRIP PICC

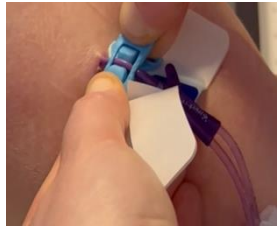

- Patient on back in comfortable position
- Inspect entry point of catheter & surrounding skin
- Disinfect skin properly and let dry

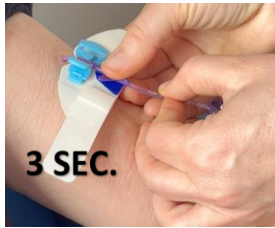

- Remove release liner of the blue triangle, place the suture wings onto the triangle and press for 3 seconds
- Place the FlexGRIP as close as possible to the catheter entry point



- Close the fixation device so that the 2 air pillows press against the catheter. Place your left thumb on the blue part of the device, press firmly
- With your right hand close the clip like a shirt button & close the extra safety buckle



- Carefully remove the release liner from the bottom of the FlexGRIP PICC and fixate on the skin
- Remove the side release liners to fully fix the adhesive to the skin



- Remove the release liner of the covering strip and close it over the suture wings



- Cover with transparent dressing
- Write the date on the transparent dressing as reminder to replace after max 7 days
- End result: FlexGRIP PICC properly placed



Instructional video

